

Please post this sheet somewhere you'll see it regularly, such as the inside of a cabinet door near your VistaPure system.

DAILY

Turn off the system at the end of each work day by closing the valves on the system board, water storage tank, and autoclave wand.

MONTHLY

Check water quality with the included TDS meters, and record the data in the chart in the Owner's Manual.

ANNUALLY

Replace at least the Prefilter (R3251) and Polishing Filter (R3255). Replace other filters only as needed, as indicated by TDS test.

Check the air pressure in the water storage tank. Air pressure should be between 6.0 psi (41 kPa) and 7.5 psi (52 kPa). See instructions on page 12 of Owner's Manual or online.

Sanitize the water storage tank and faucet using non-scented chlorine bleach and included syringe. See instructions on page 13 of Owner's Manual or online.

Filter Life & Replacement Thresholds

- 1 Prefilter (R3251)**
Approximately one year (change at least annually).
- 2 Deionization Filter (R3252)**
4 months to 3 years, depending on volume of usage and quality of local water supply. Change BOTH deionization filters whenever the system-mounted TDS meter "OUT" displays a number equal to or greater than 4 parts per million (ppm).
- 3 Deionization Filter (R3253)**
4 months to 3 years.
Replace at the same time as R3252.
- 4 Hyperfiltration Element (R3254)**
3-5 years, depending on the volume of usage and quality of local water supply. Replace the hyperfiltration filter whenever the system-mounted TDS meter "IN" exceeds 10% of the TDS of the municipal water supply TO the system, as measured with the included, handheld meter.
- 5 Polishing Filter (R3255)**
Approximately one year (change at least annually).



Need help?

Visit our searchable support database online at vrg.support.



Want a friendly reminder?

When you replace a filter, let us know online at vrg.support/filters. We'll email you when it's likely time to reorder from your favorite dental dealer.